HISTOSONDA ALFA FETOPROTEINA (AFP)

Referencia: CBM-0023-R5

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HISTOSONDA ALPHA FETAL PROTEIN (AFP)

Reference: CBM-0023-R5

Assay with one vial for 5 individual reactions. 65µl for reaction.

Product classification:

*EU countries
For In Vitro Diagnostics

INTRODUCTION

Alpha Fetal Protein is the most abundant seric protein in the fetus which demonstrates great similarity to the physical properties of serum albumin. In the adult, the seric concentration of AFP is very low, apart from when the patient presents with a hepatocarcinoma or teratoma.

The genes that code for AFP and serum albumin are syntenic genes that are situated on the long arm of chromosome 4.

After birth the organism inverts the production of AFP so that it declines rapidly and also considerably increases the production of serum albumin.

There are commercial antibodies against AFP for the diagnosis of tumors that produce this protein but they almost always produce so much tissue background that a clear view becomes impossible.

Histosonda Alpha Fetoprotein is a single stranded DNA fragment of 564 nucleotides that is targeted against the RNA of AFP.

Its use is centered on the diagnosis of primitive liver tumors and embryonic tumors of the gonads.

Up to 70% of hepatocarcinomas are producers of AFP. The use of this probe combined with Histosonda Serum Albumin permits a correct diagnosis of primitive liver tumors and their differential diagnosis with metastatic tumors in the liver.

The embryonic tumors of the gonads and midline can show an important degree of complexity and distinct levels of differentiation not well recognisable with conventional histological techniques. Histosonda Alpha Fetal Protein permits the identification of the number and localization of cells producing AFP in a tissue section.

INTENDED USE

For use in In Vitro Diagnosis.

Histosonda AFP is useful for the diagnosis of primitive liver tumours and embryonic tumours of the gonads.

WARNINGS AND PRECAUTIONS

Histosonda AFP has been designed for professional use in In Vitro Diagnosis and must be manipulated by qualified and accordingly trained personnel. In order to obtain the best results, the instructions contained in the manual must be followed. Any change to the indicated temperatures, times or any other step of the process can lead to poor results. This product contains sodium azide as a preservative (NaN₃). Sodium azide is below the threshold level of 0,1% (w/v) of the total, and therefore it is not considered to be a hazardous substance. Small quantities may be eliminated in the waste water system.

COMPONENTS

1 tube with the necessary amount of Histosonda AFP for developing 5 single reactions. Histosonda AFP consists of a segment of single-stranded DNA complementary to expressed RNA with a length of 564 nucleotides. This probe has been labeled with Digoxigenin.

STORAGE CONDITIONS

Supplied reagents are stored at 3-7°C until expiration date. Do not use after expiration date.

SAMPLES

Any paraffin block section in which Alpha Fetal Protein RNA presence is to be studied. Block sections of formalin fixed paraffin embedded tissues. Sections of 4-6 micrometers in width are sufficient to conduct the study.

INTERPRETATION OF RESULTS

Samples in which Alpha Fetal Protein expression is observed will show a brownish color in the cell cytoplasm, which will contrast over the blue-violet background given by hematoxylin staining. The pathologist will evaluate the results according to their experience, drawing conclusions from the staining of the sample, in parallel with the staining observed in the positive and negative controls.

ASSAY LIMITATIONS

Histosonda AFP has been optimized to detect RNA expression in formalin-fixed, paraffin-embedded tissues. Its use is not recommended for other types of samples or preparation techniques.

The correct operation of this product has been validated using the protocols indicated in the instructions manual. The use of other procedures or the modification of the recommended protocols may lead to erroneous results.

The results from this assay must be evaluated by the pathologist in combination with the rest of available patient clinical data.

In order to obtain optimal and reproducible results it is important to rigorously maintain the time and temperature conditions indicated in the procedure.

PROCEDURE

For Digoxigenin-labeled probes

BASIS OF THE METHOD

Chromogenic In Situ Hybridization (CISH) is a technique used to determine the presence of a DNA or RNA sequence or to study gene expression, as well as for the simultaneous evaluation of tissue morphology by white light microscopy. A labeled probe of known sequence hybridizes with its target in the tissue being studied. This hybridization is then detected by an immunohistochemical process.

The Histosondas targets are messenger RNA and they are therefore designed to detect and visualise gene expression.

HistoSonda Protocol

Needed accessories for this protocol and not provided with this product:

Proteinase K, Anti-Digoxin Antibody, anti-mouse HRP commercial polymer, Diaminobenzidine (DAB) solution

For further important details including FAQs and Troubleshooting please ask for the complete HistoSonda Technical Manual at the email address: techserv@cenbimo.com

Before starting pre-heat the humid chamber to 62°C.

1a. Deparaffinization

- Heat slides at 62°C for 10mins using a hot plate or incubator.
- Immerse slides in:
 - Xylene: 10 mins
 - Xylene: 5mins
 - Absolute alcohol (ethanol or isopropanol): 1min X 2
 - Alcohol 96%: 1min X 3
 - Methanol containing 0.3% H₂O₂ or 3% H₂O₂ only: 5mins
- Wash well with distilled water, leave standing in water.

1b. Inhibition of unspecific DNA binding (optional)

Generally this step is not necessary unless the tissue to be hybridized contains a great quantity of polymorphonuclear eosinophil leukocytes (generally **bone marrow** and **gastric tissues**).

- Place slides in boiling distilled water for **30 seconds** and immediately transfer to distilled water at room temp.

! Important: excess heat treatment will result in background staining. Remove slides immediately. !

2. Deproteinization

- Prepare a dilution of any commercial Proteinase K at a work concentration of 0,033mg/ml in PBS pH 7,4.
- Remove excess water from individual slides with tissue paper.

- Cover tissue sections with the 200µl of Proteinase K and incubate in a humid chamber for exactly 10mins at room temp.
- Wash well with distilled water.
- Transfer to PBS pH 7.4 for 2mins.

! (Important: if the slides have been boiled in the microwave previously only use Proteinase K for 5 minutes as the heat treatment will have significantly sensitized the tissue sections. **Bone marrows require a Proteinase K digestion of 20min at 55°C after heat treatment** !

3. Incubation with the probe

- Remove excess buffer from sections as described previously.
- Add 65µl of probe to the tissue section ensuring the entire section is completely covered and avoiding air bubbles.
- Place the slides in a horizontal position in a humid chamber and close well.
- Incubate at 62°C for 1hr.

4. Washing the probe

! The probe solution is very viscous. It is very important to wash the slides well to ensure the probe solution is completely removed.!

- Wash the surface of the section vigorously with PBS pH 7.4 to remove the probe.
- Agitate in PBS for 5mins.

5. Revealing the probe

Protocol for manual revealing of the probe.

- Remove excess buffer from sections as before.
- Cover sections with 100µl of Anti-Digoxin and incubate in a humid chamber for 30mins at room temp.
- Wash vigorously with PBS pH 7.4 agitate in PBS for 1min.
- Remove excess buffer from sections.
- Drop commercial HRP polymer (not provided with this product) that binds the primary antibody used before over the sections (enough to cover the tissue 50-100µl) and incubate in a humid chamber following the manufacturer's instructions.
- Wash vigorously with PBS pH 7.4 , agitate in PBS 1min.
- Remove excess buffer from sections and apply commercial Diaminobenzidine (DAB) following the manufacturer's instructions.
- Wash with distilled water.
- Counter stain the sections briefly (2-3secs) with Harris hematoxylin diluted 50% in distilled water.
- Wash with distilled water, dehydrate and cover slip following normal laboratory protocols.

Emission date: 23/08/2016

Manufacturer: CENBIMO S.L.
C/Doctor Iglesias Otero s-n
27004-Lugo, Spain