## HISTOSONDA CALCITONINA

Referencia: CBM-0017-R5

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## HISTOSONDA CALCITONIN

Reference: CBM-0017-R5

Assay with one vial for 5 individual reactions. 65µl for reaction.

Product classification:  
\*EU countries  
\*For In Vitro Diagnostics\*

## INTRODUCTION

Calcitonin is a peptide of 32 amino acids and 3.4 kDa produced by the thyroid parafollicular cells or C cells.

Calcitonin is coded by a gene named calcitonin gene-related peptide (CGRP) that also includes the code for a peptide related to calcitonin but with a completely different physiological function. The production of one or the other peptide is due to a splicing alternative.

The principle physiological function of this hormone is to produce a decrease in the blood concentration of calcium and phosphates producing an incorporation of these ions into the bones.

The tumoral degeneration of C cells produces a Thyroid Medullary Carcinoma also known as Thyroid Parafollicular Carcinoma.

Approximately 70% of Thyroid Medullary Carcinomas are sporadic and approximately 30% are familial and inheritable within the context of MEN 2A. MEN 2A patients can also demonstrate C cell hyperplasia before developing a tumor.

## INTENDED USE

For use in In Vitro Diagnosis.  
Histosonda Calcitonin is useful for the diagnosis of Thyroid Medullary Carcinomas and C cell Hyperplasia.

## WARNINGS AND PRECAUTIONS

Histosonda Calcitonin has been designed for professional use in In Vitro Diagnosis and must be manipulated by qualified and accordingly trained personnel. In order to obtain the best results, the instructions contained in the manual must be followed. Any change to the indicated temperatures, times or any other step of the process can lead to poor results. This product contains sodium azide as a preservative (NaN<sub>3</sub>). Sodium azide is below the threshold level of 0,1% (w/v) of the total, and therefore it is not considered to be a hazardous substance. Small qu

## COMPONENTS

1 tube with the necessary amount of Histosonda Calcitonin for developing 5 single reactions.  
Histosonda Calcitonin consists of two segments of single-stranded DNA complementary to expressed RNA with lengths of 167 and 143 nucleotides. This probe has been labeled with Digoxigenin.

## STORAGE CONDITIONS

Supplied reagents are stored at 3-7°C until expiration date. Do not use after expiration date.

## SAMPLES

Any paraffin block section in which Calcitonin RNA presence is to be studied. Sections of 4-6 micrometers in width are sufficient to conduct the study. Preferably, the cut should be recent (no more than thirty days old). Assay results are not affected by block age. Studies have been carried out in the manufacturer's laboratories using 20 year old paraffin blocks with optimal results.

## INTERPRETATION OF RESULTS

Samples in which Calcitonin expression is observed will show a brownish color in the cell cytoplasm, which will contrast over the blue-violet background given by hematoxylin staining. The pathologist will evaluate the results according to their experience, drawing conclusions from the staining of the sample, in parallel with the staining observed in the positive and negative controls.

## ASSAY LIMITATIONS

Histosonda Calcitonin has been optimized to detect RNA expression in formalin-fixed, paraffin-embedded tissues. Its use is not recommended for other types of samples or preparation techniques.

The correct operation of this product has been validated using the protocols indicated in the instructions manual. The use of other procedures or the modification of the recommended protocols may lead to erroneous results.

The results from this assay must be evaluated by the pathologist in combination with the rest of available patient clinical data.

In order to obtain optimal and reproducible results it is important to rigorously maintain the time and temperature conditions indicated in the procedure.

## BIBLIOGRAPHY

1) Eloy C, MD. et al. Tumor-in-Tumor of the Thyroid With Basaloid Differentiation: A Lesion With a Solid Cell Nest Neoplastic Component? Int J Surg Pathol (2011) 19(2) 276-280

## PROCEDURE

For Digoxigenin- labeled probes

## BASIS OF THE METHOD

Chromogenic In Situ Hybridization (CISH) is a technique used to determine the presence of a DNA or RNA sequence or to study gene expression, as well as for the simultaneous evaluation of tissue morphology by white light microscopy. A labeled probe of known sequence hybridizes with its target in the tissue being studied. This hybridization is then detected by an immunohistochemical process.

The Histosondas targets are messenger RNA and they are therefore designed to detect and visualise gene expression.

## HistoSonda Protocol

**Needed accessories for this protocol and not provided with this product:**

Proteinase K, Anti-Digoxin Antibody, anti-mouse HRP commercial polymer, Diaminobenzidine (DAB) solution

**For further important details including FAQs and Troubleshooting please ask for the complete HistoSonda Technical Manual at the email address: techserv@cenbimo.com**

Before starting pre-heat the humid chamber to 62°C.

### 1a. Deparaffinization

1. Heat slides at 62°C for 10mins using a hot plate or incubator.
2. Immerse slides in:
  - a. Xylene: 10 mins
  - b. Xylene: 5mins
  - c. Absolute alcohol (ethanol or isopropanol): 1min X 2
  - d. Alcohol 96%: 1min X 3
  - e. Methanol containing 0.3% H<sub>2</sub>O<sub>2</sub> or 3% H<sub>2</sub>O<sub>2</sub> only: 5mins
3. Wash well with distilled water, leave standing in water.

### 1b. Inhibition of unspecific DNA binding (optional)

*Generally this step is not necessary unless the tissue to be hybridized contains a great quantity of polymorphonuclear eosinophil leukocytes (generally bone marrow and gastric tissues).*

1. Place slides in boiling distilled water for **30 seconds** and immediately transfer to distilled water at room temp.

**! Important:** excess heat treatment will result in background staining. Remove slides immediately. !

### 2. Deproteinization

1. Prepare a dilution of any commercial Proteinase K at a work concentration of 0,033mg/ml in PBS pH 7.4.
2. Remove excess water from individual slides with tissue paper.
3. Cover tissue sections with the 200µl of Proteinase K and incubate in a humid chamber for exactly 10mins at room temp.
4. Wash well with distilled water.
5. Transfer to PBS pH 7.4 for 2mins.

**! (Important:** if the slides have been boiled in the microwave previously only use Proteinase K for 5 minutes as the heat treatment will have significantly sensitized the tissue sections. **Bone marrows require a Proteinase K digestion of 20min at 55°C after heat treatment!** !

### 3. Incubation with the probe

1. Remove excess buffer from sections as described previously.
2. Add 65µl of probe to the tissue section ensuring the entire section is completely covered and avoiding air bubbles.
3. Place the slides in a horizontal position in a humid chamber and close well.
4. Incubate at 62°C for 1hr.

### 4. Washing the probe

**! The probe solution is very viscous. It is very important to wash the slides well to ensure the probe solution is completely removed.!**

1. Wash the surface of the section vigorously with PBS pH 7.4 to remove the probe.
2. Agitate in PBS for 5mins.

### 5. Revealing the probe

*Protocol for manual revealing of the probe.*

1. Remove excess buffer from sections as before.
2. Cover sections with 100µl of Anti-Digoxin (not provided with this product) following manufacturer's instructions and incubate in a humid chamber for 30mins at room temp.
3. Wash vigorously with PBS, pH 7.4 agitate in PBS for 1min.
4. Remove excess buffer from sections.
5. Drop commercial HRP polymer (not provided with this product) that binds the primary antibody used before over the sections (enough to cover the tissue 50-100µl) and incubate in a humid chamber following the manufacturer's instructions.
6. Wash vigorously with PBS pH 7.4, agitate in PBS 1min.
7. Remove excess buffer from sections and apply commercial Diaminobenzidine (DAB) (not provided with this product) following the manufacturer's instructions.
8. Wash with distilled water.
9. Counter stain the sections briefly (2-3secs) with Harris hematoxylin diluted 50% in distilled water.
10. Wash with distilled water, dehydrate and cover slip following normal laboratory protocols.

Emission date: 23/08/2016

**Manufacturer: CENBIMO S.L.**  
**C/Doctor Iglesias Otero s-n**  
**27004-Lugo, Spain**