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HISTOSONDA VASOACTIVE INTESTINAL PEPTIDE (VIP)

Reference: CBM-0015-R5

Assay of one vial for 5 individual reactions. 65µl for reaction.

Product classification:
*EU countries
"For In Vitro Diagnostics"

INTRODUCTION

VIP is produced by some of the hypothalamic neurons and by the intestinal enteric plexus neurons. VIP producing cells do not exist in the pancreas but surprisingly, there exist pancreatic insular tumors that produce VIP and are thus named VIPOMAS. These tumors are situated in the tail of the pancreas and also in the duodenum in children. Some pheochromocytomas have also been described in the literature as producing this peptide.

Patients with VIPOMAS demonstrate a syndrome characterized by watery diarrhea, hypokalemia and achlorhydria, known as Verner-Morrison Syndrome. Approximately 80% of VIPOMAS have a malignant behavior.

HISTOSONDA VIP consists of a single stranded DNA fragment with a length of 533 nucleotides which has been labeled with digoxigenin.

INTENDED USE

For use in In Vitro Diagnosis.

Histosonda Vasoactive Intestinal Peptide is useful for the study and classification of neuroendocrine tumors of the pancreas and digestive apparatus and their metastases even in an absence of a clinically evident Endocrine Syndrome.

WARNINGS AND PRECAUTIONS

Histosonda Vasoactive Intestinal Peptide has been designed for use in In Vitro Diagnosis for professional use and must be manipulated by qualified and accordingly trained personnel.

In order to obtain the best results, the instructions contained in the manual must be followed. Any change of the indicated temperatures or times, or of any other step of the process can lead to poor results.

COMPONENTS

1 tube with the necessary amount of Histosonda Vasoactive Intestinal Peptide for developing 5 single reactions.

Histosonda Vasoactive Intestinal Peptide consists of a single stranded DNA fragment with a length of 533 nucleotides which has been labeled with digoxigenin.

STORAGE CONDITIONS

Supplied reagents are stored at 3-7°C until expiration date. Do not use after expiration date.

SAMPLES

Any paraffin block section in which Vasoactive Intestinal Peptide RNA presence is to be studied. Sections of 4-6 micrometers in width are sufficient to conduct the study. Preferably, the cut should be recent (no more than thirty days old). Assay results are not affected by block age. Studies have been carried out in the manufacturer's laboratories using 20 year old paraffin blocks with optimal results.

INTERPRETATION OF RESULTS

Samples in which Vasoactive Intestinal Peptide expression is observed will show a brownish color in the cell cytoplasm, which will contrast over the blue-violet background given by hematoxylin staining. The pathologist will evaluate the results according to their experience, drawing conclusions from the staining of the sample, in parallel with the staining observed in the positive and negative controls.

ASSAY LIMITATIONS

Histosonda Vasoactive Intestinal Peptide has been optimized to detect RNA expression in formalin-fixed,

paraffin-embedded tissues. Its use is not recommended for other types of samples or preparation techniques.

The correct operation of this product has been validated using the protocols indicated in the instructions manual. The use of other procedures or the modification of the recommended protocols may lead to erroneous results.

The results from this assay must be evaluated by the pathologist in combination with the rest of available patient clinical data.

In order to obtain optimal and reproducible results it is important to rigorously maintain the time and temperature conditions indicated in the procedure.

PROCEDURE

For Digoxigenin- labeled probes

BASIS OF THE METHOD

Chromogenic In Situ Hybridization (CISH) is a technique used to determine the presence of a DNA or RNA sequence or to study gene expression, as well as for the simultaneous evaluation of tissue morphology by white light microscopy. A labeled probe of known sequence hybridizes with its target in the tissue being studied. This hybridization is then detected by an immunohistochemical process.

The Histosondas targets are messenger RNA and they are therefore designed to detect and visualise gene expression.

HistoSonda Protocol

For further important details including FAQs and Troubleshooting please refer to the complete HistoSonda Technical Manual.

Before starting pre-heat the humid chamber to 62°C.

1. Deparaffinization

- Heat slides at 62°C for 10mins using a hot plate or incubator.
- Immerse slides in:
 - Xylene: 10 mins
 - Xylene: 5mins
 - Absolute alcohol (ethanol or isopropanol): 1min X 2
 - Alcohol 96%: 1min X 3
 - Methanol containing 0.3% H₂O₂ or 3% H₂O₂ only: 5mins
- Wash well with distilled water, leave standing in water.

1b. Inhibition of unspecific DNA binding (optional)

Generally this step is not necessary unless the tissue to be hybridized contains a great quantity of polymorphonuclear eosinophil leukocytes (generally bone marrow and gastric tissues).

- Place slides in boiling distilled water for 30 seconds and immediately transfer to distilled water at room temp.

! Important: excess heat treatment will result in background staining. Remove slides immediately. !

2. Deproteinization

- Prepare a dilution of any commercial Proteinase K at a work concentration of 0,033mg/ml in PBS pH 7,4.
- Remove excess water from individual slides with tissue paper.
- Cover tissue sections with the 200µl of Proteinase K and incubate in a humid chamber for exactly 10mins at room temp.
- Wash well with water.
- Transfer to PBS pH 7.4 for 2mins.

! (Important: if the slides have been boiled in the microwave previously only use Proteinase K for 5 minutes as the heat treatment will have significantly sensitized the tissue sections. **Bone marrows require a Proteinase K digestion of 20min at 55°C after heat treatment) !**

3. Incubation with the probe

- Remove excess buffer from sections as described previously.
- Add the 65µl of probe to the tissue section ensuring the entire section is completely covered and avoiding air bubbles.
- Place the slides in a horizontal position in a humid chamber and close well.
- Incubate at 62°C for 1hr.

- If no suitable humid chamber is available drop the 65µl of probe over a 24 X 50mm coverslip and place over the tissue section. Seal the edges of the coverslip with rubber cement. Incubate in a 62°C incubator or over a hot plate for 1 hour. After incubation carefully remove the rubber cement and coverslip and continue the protocol.

4. Washing the probe

! The probe solution is very viscous due to large quantities of dextran. It is very important to wash the slides well to ensure the probe solution is completely removed.!

- Wash the surface of the section vigorously with PBS pH 7.4 to remove the probe.
- Agitate in PBS for 5mins.

5. Revealing the probe

Protocol for manual revealing of the probe. Alternatively, an automatic immunohistochemistry apparatus may be used.

- Remove excess buffer from sections as before.
- Cover sections with 100µl of Anti-Digoxin (not provided with this product) and incubate in a humid chamber for 30mins at room temp.
- Wash vigorously with PBS, pH 7.4 agitate in PBS for 1min.
- Remove excess buffer from sections.
- Drop commercial HRP polymer (not provided with this product) that binds the primary antibody used before over the sections (enough to cover the tissue 50-100µl) and incubate in a humid chamber following the manufacturer's instructions.
- Wash vigorously with PBS, agitate in PBS 1min.
- Remove excess buffer from sections and apply commercial Diaminobenzidine (DAB) following the manufacturer's instructions.
- Wash with water.
- Counter stain the sections briefly (2-3secs) with Harris hematoxylin diluted 50% in water.
- Wash with water, dehydrate and coverslip following normal laboratory protocols.

Emission date: 23/08/2016

Manufacturer: CENBIMO S.L.
C/Doctor Iglesias Otero s-n
27004-Lugo, Spain